


Easy Implant Removal

EFR KIT

Bone loss-less fixture removal
Quick & Simple removal by two(2) Stage
Above 80% removal success

OSSTEM[®]
IMPLANT

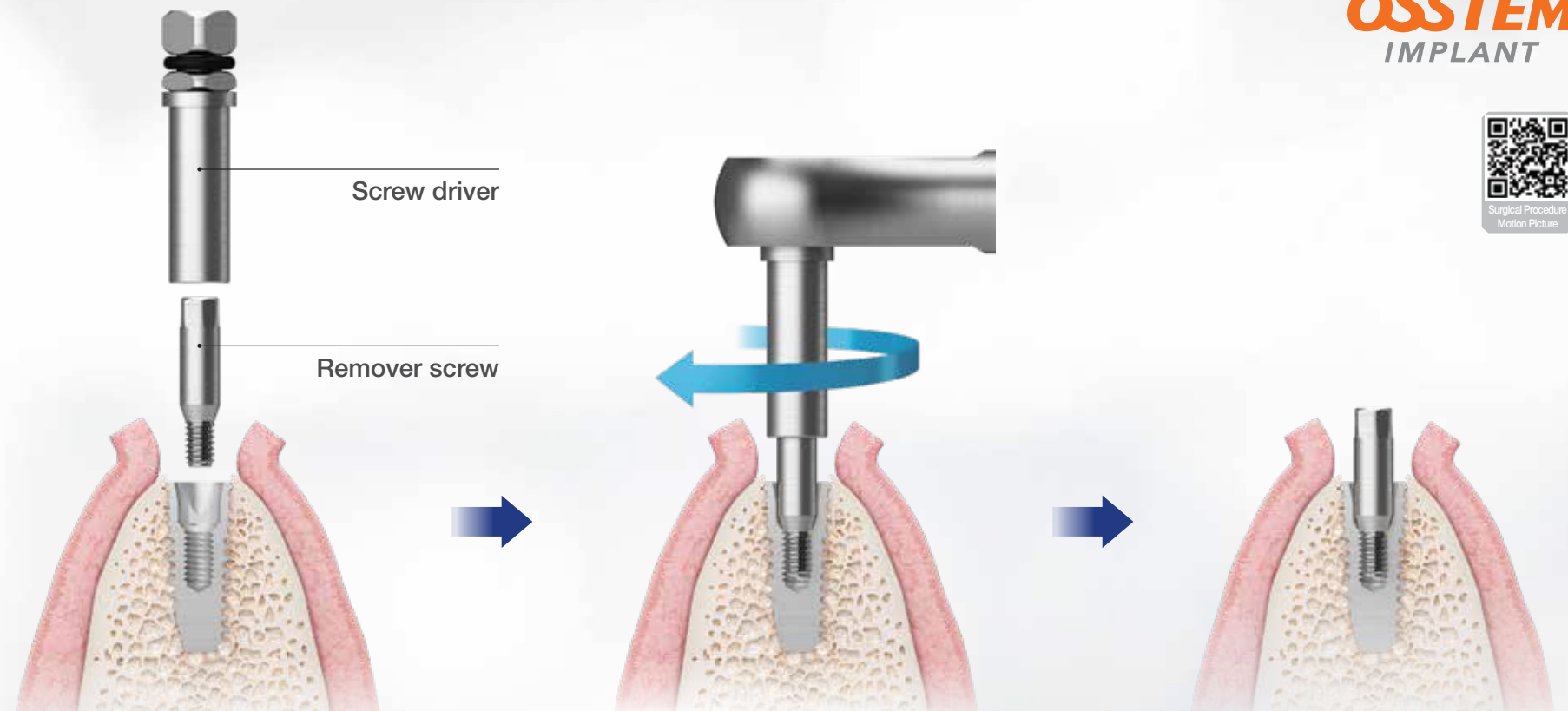


1. Select remover screw
(Refer to the remover screw selecting guide)

2. Clockwise tightening by torque wrench
through the screw driver
(mini 80Ncm, regular 100Ncm)

3. Complete the removal screw
installation into the fixture

Remover Screw Selection Guide

Fixture Condition Check	Normal	Crack	Partial fracture	Fracture to internal hex connection area (Only internal screw thread remained)
Remover Screw Selection	<p>Normal Remover screw connection</p>			<p>Fracture Remover screw connection</p>

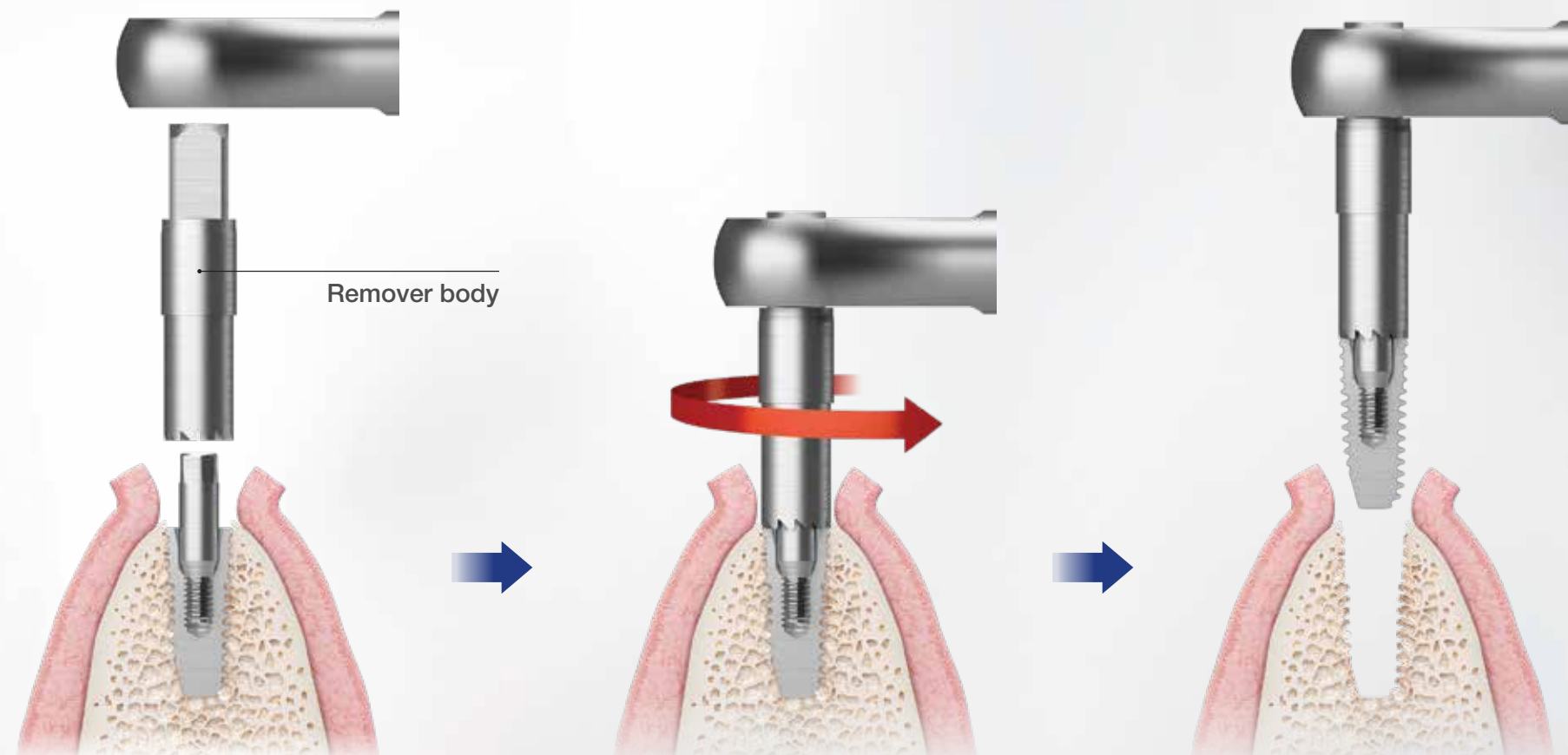

4. Removal body selection

5. Anti-clockwise screwing by torque wrench
through the removal body
(Maximum 400Ncm engagement possible)

6. Bone loss-less fixture removal by
strong adhesion property between fixture
& removal body